How to use your new Rehband Thermal Shorts - 7380*, 8041*, 8131*, 8141*, 8231*, 8241*

*different colours, sizes, thicknesses, sides

(EN)

Function

Provides support, compression, and warmth to the lower back and hip area. Increases blood circulation, relieves pressure, and improves muscle coordination in that region.

Application

Pull the shorts to a comfortable fit

Important Information

- 1) Use this product only for its intended purpose. Do not make any improper modifications to the product.
- 2) If you have any questions regarding its use, please contact your local retailer.
- 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- 5) Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C/248 °F.
- 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.

Size selection

Determine your size according to the size chart.

Disposal

Dispose the product in accordance with national regulations.

Material

Foam: SBR/neoprene or neoprene (see textile label) Cover: polyamide and/or polyester (see textile label)

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