
# How to use your new Rehband QD Knitted Back-Support – 6902\*

\*different sizes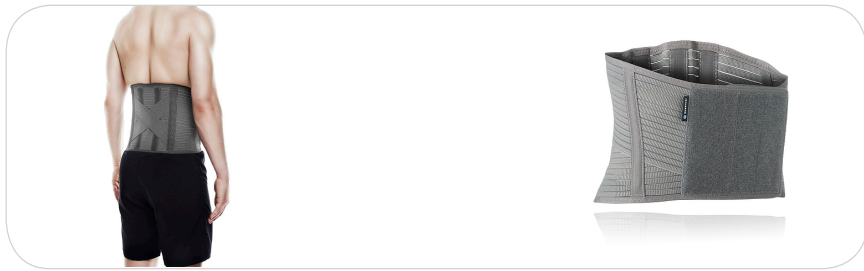

UK

## Material

Polyamide, polyester, spandex, ABS-splint

## Application

1. If needed, adjust the size with the extension piece.
2. Place the support around the lower back.
3. Close the Velcro closure in the front.

## Important Information

- 1) If you have questions regarding application, please contact your local dealer.
- 2) The product is not flameproof, avoid temperature above 120°C/ 248°F.
- 3) The function of the product must not be changed or compromised, the product should only be used for its original.
- 4) The product is made for possible all day use. For your safety though, see point 5.
- 5) If you are allergic to heat, we advise you to not use the product for more than 3-4 hours at a time.
- 6) Ensure that the support / orthosis is not over-tightened to avoid risk of local pressure and constriction of blood vessels and nerves. Verify the correct positioning and fit of the support / orthosis.
- 7) Do not use the product on more than one patient to avoid skin irritation, formation of eczema or infections.

## Washing Instructions

- 1) Close all Velcro.
- 2) Mesh laundry bag is recommended.
- 3) Stretch product while wet and let dry.

## Liability

The manufacturer's warranty applies only if the device has been used under the conditions and for the purposes described. The manufacturer recommends that the device be used and maintained according to the instructions for use.

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**Rehband Limited**Villa Elias • 8740 Lasa/Pafos • Cyprus