How to use your new Rehband Cohesive Tape - 0183*

*different colours

EN

Function

Cohesive tapes are self-adhesive bandages that do not require glue. They are suitable for securing dressings, supporting joints, providing light compression, or stabilization.

Application

- 1) Wrap the tape evenly and without creases around the desired area.
- 2) Apply the tape with slight tension but avoid excessive pressure to prevent restricting blood circulation.

Important Information

- 1) Use this product only for its intended purpose. Do not make inappropriate modifications to the product.
 - 2) If you have any questions about the application, please contact your local dealer.
 - 3) Use the tape only for short-term applications and regularly check blood circulation.
 - 4) Remove the tape immediately in case of skin irritation, pain, or blood circulation problems.
- 5) Do not use the product on open wounds.

Safety and Care Instructions

- 1) Store the product in a dry place at room temperature. Avoid direct sunlight.
- 2) Do not use a product that is no longer functional, worn out, or damaged.

Disposal

Dispose the product in accordance with national regulations.

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