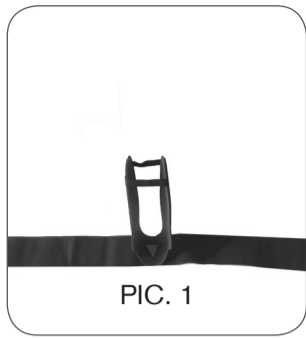
How to use your new Rehband Ankle Support – 127* (7761*)

*different colors, sizes, thicknesses, sides

PIC. 1

PIC. 2

PIC. 3

PIC. 4

PIC. 5

Function

Gives support, compression and warmth to the ankle joint. Increases blood circulation, relieves pressure and improves muscle coordination in the area. Stabilises the joint area.

Application

- 1) Put the support on the floor with the arrow inside pointing forwards and the straps to the sides (pic. 1).
- 2) Pull the support over the foot (pic. 2).
- 3) Take the inner strap and make a loop around the ankle with desired pressure. Fasten the strap to the loop (pic. 3).
- 4) Take the other strap and make a loop around the ankle to stabilise the support. Fasten the strap to the loop (pic. 4).
- 5) Pull the stocking over the support (pic. 5).

Important Information

- 1) Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- 2) If you have questions regarding application, please contact your local dealer.
- 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- 5) Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C/248 °F.
- 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.
- 5) Fasten all velcro closures.
- 6) Products with velcro fasteners require occasional dirt/dust removal.

Size selection

- 1) Do the measurements as instructed on the packaging.
- 2) Determine the size of the support (see size chart).

Disposal

Dispose the product in accordance with national regulations.

Material

Cover: polyamide, elastane (see textile label)
Stabilizer: ethylene-vinyl acetate (EVA)

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