How to use your new Rehband Support – 113* (7730*, 7930*), 123* (7732*), 133* (7792*), 7082*

*different colors, sizes, thicknesses, sides

EN

Function

Provides support, compression and warmth to the back. Increases blood circulation, relieves pressure and improves muscle coordination in the area. 133* suitable for sport activities and lifting heavy weights.

Application

- 1) Place the support on the lower back.
- 2) Hold the left end from the lower edge of the support. Take the right end and place it over the left part. Tighten until you feel the desired pressure from the support.
- 3) 123*: The level of pressure on the back can be varied by adjusting the two additional flexible straps, by pulling them either a little upwards or downwards before fastening at the stomach.
- 4) 133*: Can be used with an additional belt for extra support.

Important Information

- 1) Use this product with care and only for its intended purpose. Do not make any improper changes to the product.
- 2) If you have questions regarding application, please contact your local dealer.
- 3) Do not wear products with SBR/neoprene for more than 3-4 hours without interruption.
- 4) We advise you to not use the product that contains SBR/neoprene in case of tendency to heat allergy.
- 5) Keep the product away from open flame, embers and others sources of heat. Avoid temperature above 120 °C/248 °F.
- 6) Do not continue using a product that is no longer functional, or that is worn or damaged.

Washing Instructions

- 1) Wash in warm water at 40 °C (104 °F) with similar colours.
- 2) Use laundry bag or net.
- 3) Stretch product while wet and allow to air dry.
- 4) Clean the product regularly according to care instructions on the product care label.
- 5) Fasten all velcro closures.
- 6) Products with velcro fasteners require occasional dirt/dust removal.

Size selection

- 1) Do the measurements as instructed on the packaging.
- 2) Determine the size of the support (see size chart).

Disposal

Dispose the product in accordance with national regulations.

Material

Foam: SBR/neoprene or neoprene (see textile label)
Cover: polyamide and polyamide-elastane (see textile label)
Plastic splints (123*): polypropylene
Metal splints (133*): steel
Additional support belt (133*): Metal buckle: steel
Cover: polyester
Foam: ethylene-vinyl acetate (EVA)

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Rehband Limited